

30 mL 25x Wash Solution Concentrate for KIT 029

PRODUCT NO. **WASH 029-30**

PRESENTATION Preparation: 25x Wash Solution Concentrate
Content: 30 mL
Storage: Store at 2-8°C. Do not freeze.
Stability, undiluted: See label
Stability after dilution: 4 weeks at 2-8°C. Do not freeze.

PREPARATION Dilute the 25x Wash Solution Concentrate by pouring the total contents of the bottle (30 mL) into a 1-L graduated cylinder and add distilled or deionized water to a final volume of 750 mL. Mix thoroughly and store at 2-8°C after use.

INTENDED USE The 30 mL 25x Wash Solution Concentrate for KIT 029 is intended for use with the MBL Oligomer ELISA Kit when used with automated ELISA washing systems. The kit already contains 30 mL 25x Wash Solution Concentrate, which should be sufficient for manual washing.

INSTRUCTIONS Please refer to the instructions for use for KIT 029 MBL Oligomer ELISA Kit for further information. For *in-vitro* diagnostic use in the European Union. For research use only in the rest of the world.

CONDITIONS

All products are supplied on the understanding that they are for in vitro use only. The information and product are offered without guarantee as the ultimate conditions of use are beyond our control.